DE/EN

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**INSTRUCTIONS
FOR USE**

WheelChair-Link for AMAneo MX

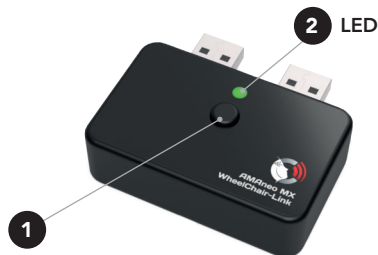
C.S.S.
MICRO SYSTEMS

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The **WheelChair-Link** allows electric wheelchairs with a built-in Bluetooth® interface to be connected to the **AMAneo MX** for the joystick's mouse function, enabling the wheelchair joystick to be used to control gaming consoles and similar devices.

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Setup / Use

Requirement: FW 1.3 **AMAneo MX**! If necessary, please update **AMAneo MX** first; see the Instructions for Use.

Plug the **WheelChair-Link** into the **AMAneo MX** (both USB ports). It will then automatically register there.

In normal mode, the **WheelChair-Link** does not actively search for a connection but only reconnects to the last known wheelchair. To pair with the wheelchair, press the pairing button ❶. This activates the search for a wheelchair for approx. 30 seconds; the LED ❷ flashes blue. Put the wheelchair into pairing mode (see the wheelchair controller's instructions).

The **AMAneo MX** display now shows all pairable wheelchairs and other devices in the environment. Please use the ▲ and ▼ buttons to select the desired wheelchair and press ⏻ to confirm. As soon as the connection is established, the search ends automatically and the device saves this input device for future reconnections. This process may take up to one minute. The LED ❷ lights up blue. Pressing the button briefly again manually ends the search.

LED Indications:

- **Green for 3 sec. after startup:** WheelChair-Link started.
- **Permanent blue:** Connected to wheelchair.
- **Flashing blue:** Pairing/scanning active.
- **Flashing red for 3 sec.:** Connection unexpectedly lost.
- **Off:** No connection, no scanning.

Button ❶ Function:

- **Press briefly:** Start pairing/scanning, then select the wheelchair in the **AMAneo MX** menu.
- **Press briefly again:** Stop pairing/scanning.
- **Press and hold for at least 3 seconds:** Delete current connections.

Notes:

- The system always prioritizes reconnecting to the most recently paired or most recently known wheelchair.
- If no wheelchair is connected, the LED ❷ is off.
- After the search time has elapsed, the search automatically turns off.

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© 2026 CSS MicroSystems GmbH